

Colostomy bag instruction

One Piece System



One Piece System Colostomy Bag

1. Use the water to clean the skin around stoma.
2. A hole have been advanced opening at the flange, the user can cut a hole at the flange.



3. Remove the protection paper from the flange.



4. Blow a little air into the bag before fixing the bag, in order to prevent the wall of the bag being stuck on. Then hold the cutting sleeve aimed at the flange by using two hands from the bottom to the top, use the finger to press it carefully and make it conglutinate absolutely.



About Drainable Bag

5. If using a drainable pouch make sure the opening or tap is securely closed by clamps or agraffe or 'Velcro' before applying the new pouch.




6. Open the end of the bag when giving out excrement, make the end of the bag aim at the urinal and then wash the end of the bag as the picture, make the end of the bag smoothly, then can reuse.



Notice

1. The user who intestines opened can use this product to incept the ordure only.
2. The 'flange' to mix this product can use 3-5 days in normal condition. It can affect the effect of the affixing if use delay, this is not good to the skin.
3. If the bag full of air, make it exhausted by using the pinhole in the upper end of the bag. Suggest use activate carbon cassette, it can wipe off the peculiar smell.
4. Once exchanged the bag and can not be used again.

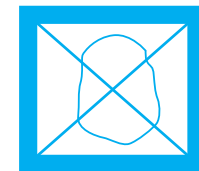

5. Don't wash the bag into the toilet or cloaca, it can not be discarded at will, pay attention to the protection of the environment.

6. Be prudent to use this product if allergic skin.

